

IndiMag[®] 2


Quick Start Guide: Firmware Setup

Further information and support

- Please make sure that the latest version of the IndiMag 2 software is installed on your device. You can check for the latest firmware version in the download section at www.indical.com/indimag2.
- IndiMag 2 Handbook: www.indical.com/indimag2
- IndiMag Pathogen IM48 Cartridge, and IndiMag Pathogen Kit w/o plastics Handbooks: www.indical.com/handbooks
- Technical assistance: support@indical.com

Run protocol


1. The main menu is the starting point of the user interface. From here all other software functions can be accessed. The main menu can always be reached by pressing the 'home' icon.

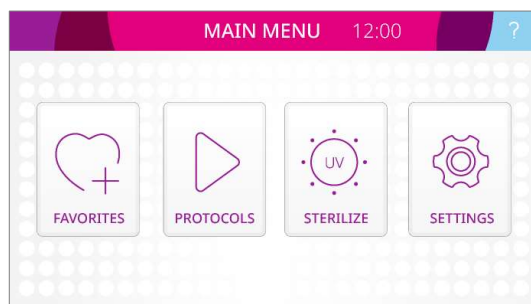

2. From the 'start / play' icon (PROTOCOLS) in the main menu, the saved extraction protocols can be accessed and run. To run an extraction process, select a protocol from the list.



3. Load the consumables into their dedicated positions in the IndiMag 2. Make sure all sample positions contain correctly installed rod covers, before starting the protocol by pressing the 'play' icon.

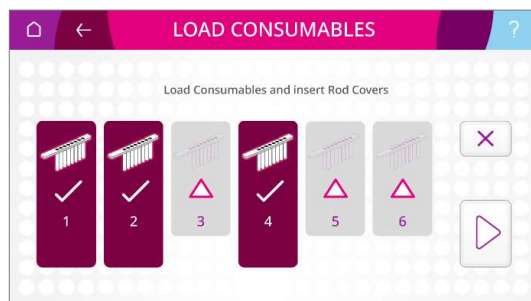

4. After the run has started, the name and the progress of the running extraction protocol is displayed on the screen. A countdown displays the remaining time until the protocol/run is completed.



Table 1. Main menu overview.












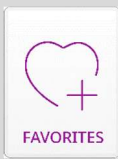


Icon	Name	Functions
	Favorite Protocol Overview	Most used protocols can be easily accessed from the 'FAVORITES' icon.
	Protocol Overview	Saved extraction protocols can be accessed and run via the 'start / play' icon (PROTOCOLS).
	UV Sterilization	UV sterilization parameters can be accessed, set, and run from the 'STERILIZE' icon.
	Settings	Device settings, language settings, time settings, protocol editor, as well as protocol import/export, firmware update, reset device and service and maintenance options can be accessed via the 'SETTINGS' icon.

Table 2. List of user interface symbols and their functions.

Icon	Meaning	Function
	Home	Returns to the main menu
	Play	Starts procedure
	Pause	Pauses progress
	Accept data	Accepts changed data
	Delete data	Deletes changed data/ deletes protocol
	Save	Saves progress / setting
	Back	Goes back one screen
	Rename	Renames protocol
	Help	Provides help and directions
	Exit/ Abort	Aborts progress without saving it

For up-to-date licensing information and product-specific disclaimers, see the respective INDICAL kit handbook or user manual. Trademarks: IndiMag® and INDICAL® (INDICAL BIOSCIENCE). Registered names, trademarks, etc., used in this document, even when not specifically marked as such, are not to be considered unprotected by law. HB-2661-EN-001 04/2024 © 2024 INDICAL BIOSCIENCE GmbH, all rights reserved.