



A single protocol for all flocktype poultry assays from INDICAL


Test components

User-friendly, color-coded reagents

Sample Diluent		Ready-to-use	Positive Control		Ready-to-use
Conjugate		Ready-to-use	Negative Control		Ready-to-use
Substrate		Ready-to-use	Wash Buffer		Dilution 1:10
Stop Solution		Ready-to-use	ELISA plate		


Test preparation

flocktype ELISA preparation

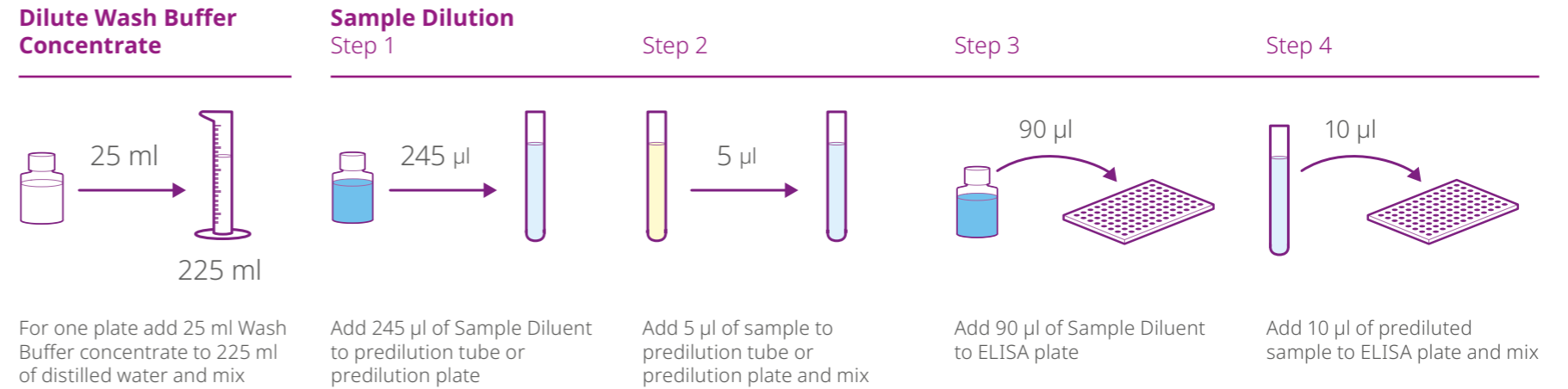

Test protocol

Harmonized and convenient indirect ELISA protocol

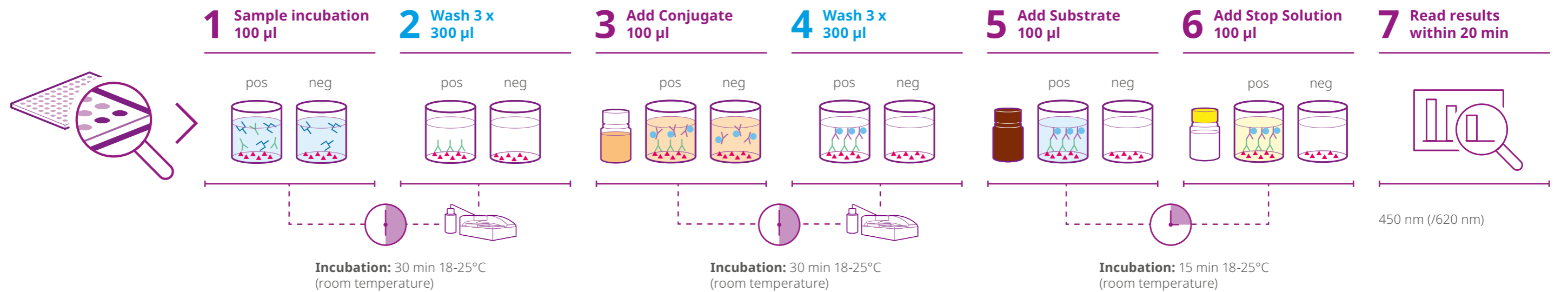

Test results

Easy result interpretation

	flocktype IBV Ab	flocktype IBDV Ab	flocktype NDV Ab	flocktype Salmonella Ab	flocktype Mycoplasma Ms Ab	flocktype AIV Ab
Cat. No.	FT274303 (5 plates)	FT274203 (5 plates)	FT275003 (5 plates)	FT275702 (2 plates)	FT274602 (2 plates)	FT274012 (2 plates)
Detects antibodies to	Infectious Bronchitis Virus	Infectious Bursal Disease Virus	Newcastle Disease Virus	S. Enteritidis & S. Typhimurium	Mycoplasma synoviae	Avian Influenza A Virus
Cut-off	Negative S/P < 0.25 Suspect - Positive S/P ≥ 0.25	S/P < 0.2 - S/P ≥ 0.2	S/P < 0.3 - S/P ≥ 0.3	S/P < 0.2 S/P ≥ 0.2 and < 0.3 S/P ≥ 0.3	S/P < 0.1 S/P ≥ 0.1 and < 0.2 S/P ≥ 0.2	S/P < 0.3 - S/P ≥ 0.3
Validation criteria	Mean OD _{PC} - Mean OD _{NC} ≥ 0.2 Mean OD _{NC} ≤ 0.2			Mean OD _{PC} ≥ 0.7 Mean OD _{NC} ≤ 0.2		



Get in touch with our experts!

Email us: support@indical.com | orders@indical.com
Call us: +49 341 124 54 0
Learn more at: www.indical.com | shop.indical.com

For up-to-date licensing information and product-specific disclaimers, see the respective handbook or user manual. Assays for veterinary use only. Product availability/distribution: outside the U.S. and Canada. Regulatory requirements vary by country, products may not be available in your geographic area. Trademarks: INDICAL®, flocktype® (INDICAL BIOSCIENCE). Registered names, trademarks, etc., used in this document, even when not specifically marked as such, are not to be considered unprotected by law. Product illustrations may differ from the actual product. IN-FLTYPE-ELISA-POSTER-EN-230809 © 2023 INDICAL BIOSCIENCE

INDICAL BIOSCIENCE GmbH
Deutscher Platz 5b
04103 Leipzig, Germany

